NEU

ANAPLASTOLOGY-CARE stellt vor:

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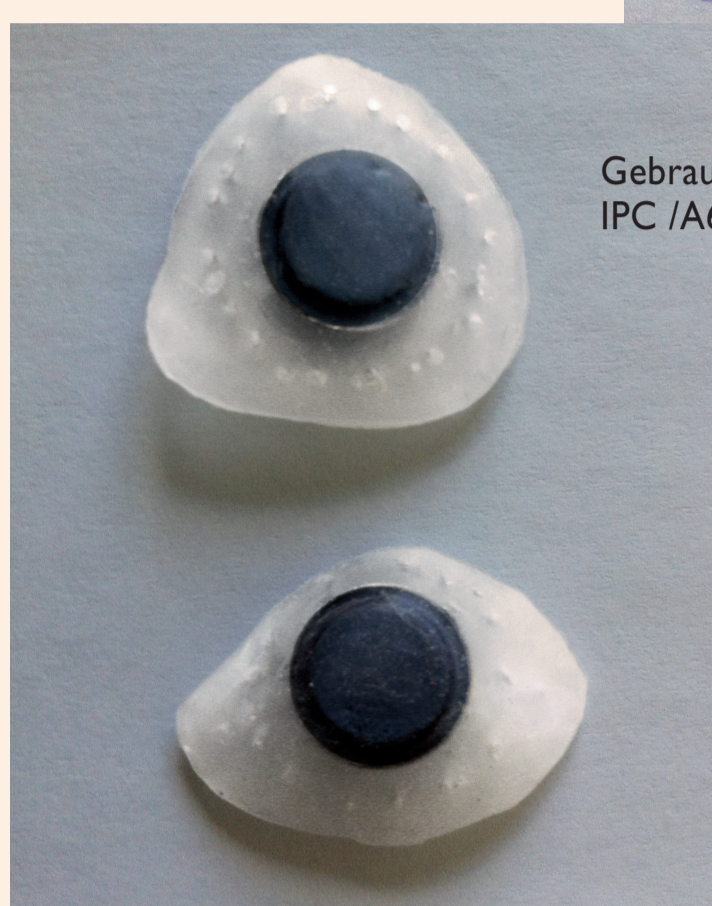
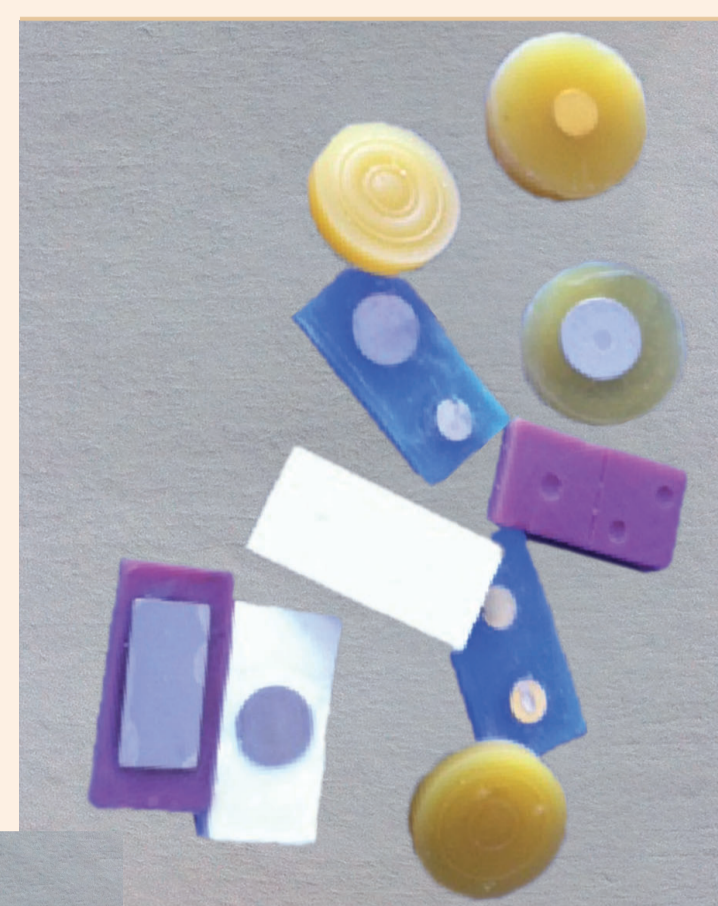
FINCRESCO® – a new anchoring technique in anaplastology

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Gebrauchsmustereintrag Nr. 202012 100 963.3
IPC /A61F 2/78

Often, insufficient osseous matter for transdermal or hypodermic magnet insertion poses a problem in the anaplastologic covering of a defect.

We would like to present an alternative approach.

We use in our FINCRESCO patented technique hermetically- and porosity-sealed, titanium encapsulated magnets and embed them in long-term silicone cushions. After production they can be sterilized according to the regulations and surgically sewn to the subcutaneous tissue at an explicit position. Perforations as they can be created by punch biopsy for example allow for tissue to grow through the implants and secure the exact position.

Once the wound has healed the manufacturing of the epithesis can start. It makes sense to determine the contact pressure which is comfortable and pain-free for the patient beforehand. Shape and color help determining the best solution.

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Example case I

In addition to the surgical rehabilitation an anaplastological treatment was also planned.

The very thin, scarred skin and the disadvantageous bone setting contraindicate the use of conventional implants. To avoid further skin irritation, adhesive-affixed epithesis are not considered a viable option. An additional handicap is posed by the loss of the right forearm in the fire accident and the left eye socket was eviscerated. For his myopia he needs 28-diopter corrective lenses.

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Two FINCRESCO per side were inserted.

This picture shows the patient with the wax model during the fitting.

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The patient after the completion of the nose and ear epithesis.

A light human hair wig additionally helps concealing.

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Example case II

While we can unfortunately only produce semi-perfect results when starting with a challenging initial situation, the results for good to very good initial situations are very aesthetic, as in the example shown of a patient with an ear defect.

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